# WOUND-CLASS

WUNDTHERAPIE-SCHUHSYSTEM

WOUND THERAPY SHOE SYSTEM

ENGINEERED IN GERMANY

MADE IN ITALY

WOUND-CLASS +

## MODELL **Saluped**

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The „Saluped“ was developed in collaboration with specialists in diabetic foot clinics. It is the optimal solution for mastering the challenges of treating diabetic foot ulcerations. Rely on specialist know-how to significantly improve your patients' chances of recovery.

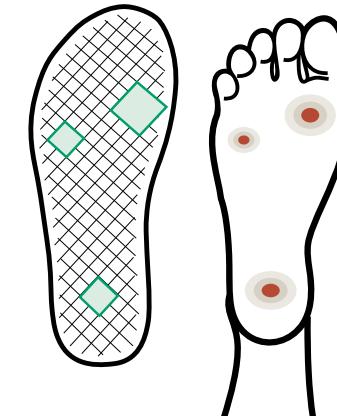
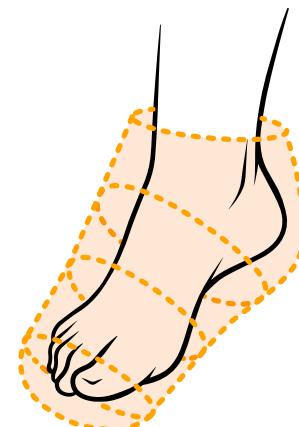
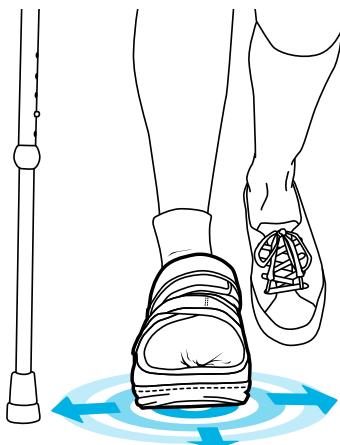
The product name „Saluped“ is a combination of the Latin words for „benefit“ and „foot“ and the American term for orthopaedics.

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The innovative sole construction of the WOUND-CLASS Saluped offers outstanding features in diabetic care:

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#### Material thermoplastic rubber

- slip resistance
- wear resistance
- water resistance

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#### Injection-moulded sole with 81% stand space

- Maximum stability
- Absolute straight edition

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#### Optimized sole profile

- Smooth at the front, minimizes the risk of stumbling
- Flat knobbed profile in the middle and outside rough grooves offer the best grip and enable easy cleaning
- Rear slats for additional support

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#### GOAL

Impaired tactile perception is a problem for many diabetics. The lack of tactile information increases the risk of injury. Stance and gait safety is therefore an important feature for adequate shoe fitting.

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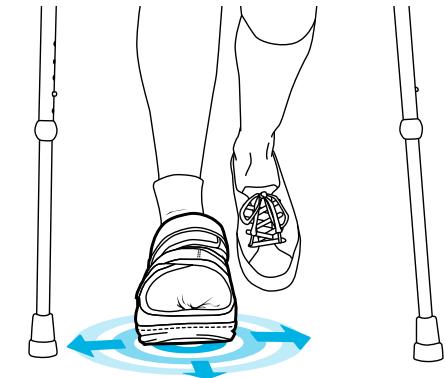
### STAND SECURITY

The Saluped offers maximum stability thanks to an ankle-high stabilising shaft that effectively restricts joint movement. Good support is also guaranteed by a perfect fit. Thanks to individual sizes from 37-46, the foot always sits perfectly in the shoe without micro-movements leading to further damage. The last shape is narrower in the heel up to size 41 to prevent slender feet from slipping out.

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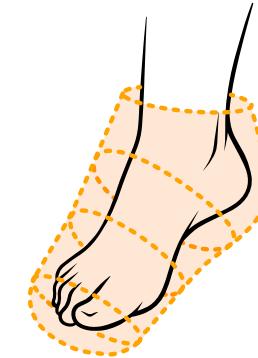
The concept of the wound therapy shoe system offers proven features in diabetic care.

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## GOAL

For diabetics with open wounds, the main task of the Saluped is to hold large-volume dressings. These are protected from slipping and external influences such as cold, moisture, splash water and pressure. Hygiene is an important requirement when treating diabetic ulcers. The Saluped can be disinfected.

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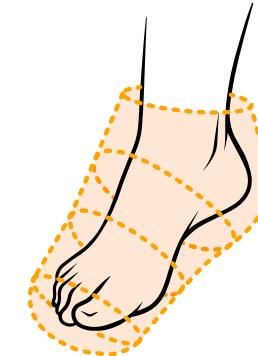
Thanks to its stretch shaft with long Velcro fasteners that can be cut off, the Saluped offers extra space to accommodate wound dressings. The shaft is comfortably padded. The modular hook-on forefoot cap protects against cold and wet splashes.

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### WEATHER PROTECTION

A protective forefoot cap can be easily easily and modularly as required.

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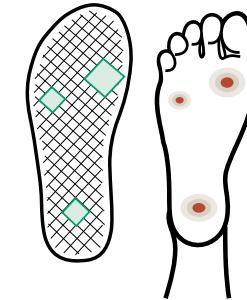
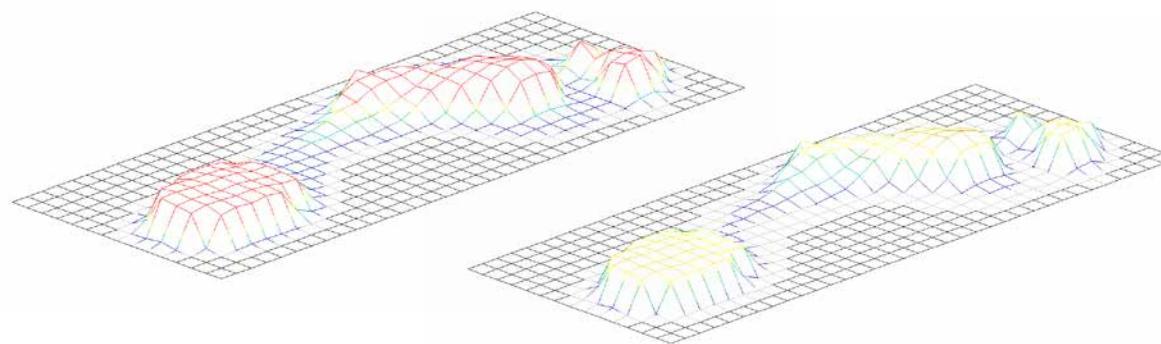
Pedographic measurements of the Saluped during outpatient foot consultations revealed significant pressure redistribution due to the sole geometry alone, without any further measures.

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### GOAL

70% of ulcerations in diabetic foot syndrome occur in the forefoot. The **outsole of the Saluped with stiffening and roll** promotes wound healing. This is because a good roll restricts the flexion of the metatarsophalangeal joints and helps the foot to quickly move forwards over the point of strain.

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The cover is breathable, antibacterial, antimicrobial, anti-odor and hypoallergic.

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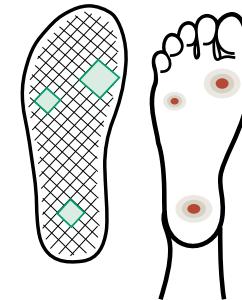
### PRESSURE RELIEF

with the honeycomb soles of the Saluped.  
Simply break out the honeycombs at specific points and then glue the  
sole to the lower layer. Quick and durable, can be customised.

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### SHOE & INSOLE

35/36 double size  
37-46 single sizes  
47/48 double size

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Can be ordered individually or in pairs as a set

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Saluped paired (same sizes) with protective forefoot cap and 3-zone relief insole with adhesive film

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## APPLICATION

- ✓ **Treatment** for non-plantar diabetic foot lesions
- ✓ **Healing** of open wounds, ulcers and boils
- ✓ **Postoperative** treatment for partial forefoot amputations

SPRINGER AKTIV AG  
Lengeder Str. 52  
D - 13407 Berlin

+49 (0)30 - 49 00 03 - 0  
 bestellung@springer-berlin.de  
 springer-berlin.de  
 shop.springer-berlin.de